



Reach for the Horizon



With
ACTIVEFOCUS™
optical design for
your active lifestyle.*

Discover the cataract treatment option that can help you explore more of your world.

Don't let cataracts leave you on the sidelines. The AcrySof® IQ ReSTOR® +2.5 D multifocal lens is designed to not only treat your cataracts, but also reduce your need for spectacles for a range of intermediate to distance vision.¹ If you have an active lifestyle that includes activities like driving, shopping, playing sports or going to the theater, ask your eye care specialist about the AcrySof® IQ ReSTOR® +2.5 D lens, a multifocal cataract treatment for your multifaceted life.

Alcon
a Novartis company

 **AcrySof® IQ ReSTOR®**
PRESBYOPIA-CORRECTING IOL

 **Advancing**
CATARACT SURGERY

*Active-lifestyle patients participate in activities that require intermediate and distance vision such as golf, tennis, house cleaning and driving.

Advanced Options for Your Needs

Multifocal lenses take vision a step further

They're your eyes, so with your eye care specialist's help, you'll decide which IOL is best for you. If you opt for basic replacement lenses — or monofocals — you may still need glasses in order to see clearly at every distance.

However, AcrySof® IQ ReSTOR® brand multifocal lenses are another story. Whether you're driving, watching TV, playing golf or taking your grandchildren to the park, AcrySof® IQ ReSTOR® brand multifocal lenses can help you experience clear vision at a range of distances without glasses.¹



Common Procedure, Excellent Results²

Cataract surgery can brighten your view of the world

A cataract occurs when the natural lens in your eye becomes cloudy, obscuring the light entering your eye and making things look out of focus.³

During cataract surgery, the cataract-affected lens in your eye is removed and replaced with a new lens, known as an IOL.



Going through life with cataracts is like viewing the world through a foggy window.



Cataract surgery is one of the most common procedures in the U.S. It's a highly successful outpatient procedure that requires only local anesthesia.²

Cataract surgery has been shown to result in the ability to read more easily, increased social activity, and reduced stress and anxiety.⁴⁻⁶

An Innovative Vision Choice

Go the distance — and everywhere in between

Have your cataracts made you lose focus on activities you love? Ask your eye care specialist if the AcrySof® IQ ReSTOR® +2.5 D multifocal lens, with its groundbreaking ACTIVEFOCUS™ optical design,¹ is a good fit for you and your active lifestyle.

ACTIVEFOCUS™ channels light to the part of your eye that processes distance vision. Additionally, it's designed to provide balanced intermediate and distance vision to help you see clearly at a wide range with less need for glasses.¹



Simulated image with basic monofocal lenses.



Simulated image with AcrySof® IQ ReSTOR® +2.5 D lenses.

Refocus on Your Lifestyle




AcrySof® IQ ReSTOR® +2.5 D IOL is designed for your active-lifestyle activities.


Intermediate



Cleaning

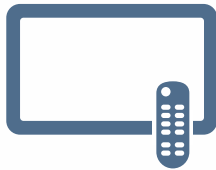

Watching TV

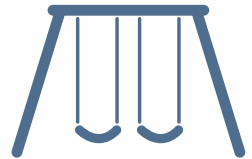

Family time


Distance



Outdoors



Driving

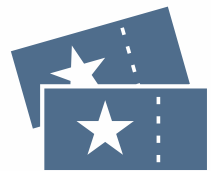

Theater



Sight-seeing



Star-gazing



Sports

Visit freedomfromcataracts.com
to learn more about investing in
your eyesight!

Investing in quality vision

Your eyesight is priceless. It plays a huge role in how you experience the world — and how you create memories with your family, your friends, your community and beyond.

Medicare or private health insurance will typically cover the costs of cataract surgery and basic replacement lenses — or monofocals — but there are usually out-of-pocket costs for multifocal IOLs.

Ask your eye care specialist if the AcrySof® IQ ReSTOR® +2.5 D multifocal lens is a good fit for you and your active lifestyle.

ACRYSOF® IQ RESTOR® FAMILY OF IOLS IMPORTANT PRODUCT INFORMATION

CAUTION: Restricted by law to sale by or on the order of a physician. **DESCRIPTION:** The AcrySof® IQ ReSTOR® Intraocular Lenses (IOLs) are artificial lenses implanted in the eye of adult patients following cataract surgery. These lenses are designed to allow for clear distance, intermediate, and near vision with the potential to be more independent of the need to use glasses for daily tasks. **WARNINGS/PRECAUTIONS:** You may experience and need to contact your eye doctor immediately if you have any of the following symptoms while using the antibiotic eye drops prescribed by your doctor: itching, redness, watering of your eye, sensitivity to light. The safety and effectiveness of the AcrySof® IQ ReSTOR® IOL has not been established in patients with eye conditions, such as an increase in eye pressure (glaucoma) or complications of diabetes in the eye (diabetic retinopathy). As with any surgical procedure, there are risks involved. These risks may include but are not limited to infection, damage to the lining of the cornea, the retinal layer which lines the inside back wall of your eye may become separated from the tissue next to it (retinal detachment), inflammation or swelling inside or outside the eye, damage to the iris (the colored diaphragm around the pupil), an increase in eye pressure that cannot be controlled by medicine and secondary surgical procedure. With this IOL, there may be a loss of sharpness of your vision that may become worse in dim light or in foggy conditions. There is also a possibility that you may have some visual effects such as rings or circles around lights at night. You may also have trouble seeing street signs due to bright lights or glare from oncoming headlights. **ATTENTION:** As with any surgical procedure, there are risks involved. Prior to surgery, ask your eye doctor to provide you with an AcrySof® IQ ReSTOR® IOL Patient Information Brochure, which will inform you of the risks and benefits associated with this IOL. Discuss any questions about possible risks and benefits with your eye doctor.

1. AcrySof® IQ ReSTOR® +2.5 D IOL Directions for Use.
2. Majka C, Carlson A. Ophthalmic pearls: cataract: when to use multifocal intraocular lenses. American Academy of Ophthalmology website. <http://www.aao.org/publications/eyenet/200609/pearls.cfm>. Accessed July 31, 2013.
3. Cataract. American Optometric Association website. <http://www.aoa.org/cataract.xml>. Accessed March 5, 2015.
4. Owsley C, McGwin G Jr, Scilley K, et al. Impact of cataract surgery on health-related quality of life in nursing home residents. *Br J Ophthalmol.* 2007; 91:1359-1364.
5. Harwood RH, Foss AJ, Osborn F, et al. Falls and health status in elderly women following first eye cataract surgery: a randomized controlled trial. *Br J Ophthalmol.* 2005;89(1):53-59.
6. Owsley C, McGwin G Jr, Sloane M, et al. Impact of cataract surgery on motor vehicle crash involvement by older adults. *JAMA.* 2002 Aug 21;288(7):841-849.

Alcon
a Novartis company



AcrySof® IQ ReSTOR®
PRESBYOPIA-CORRECTING IOL



Advancing
CATARACT SURGERY